



CT SCANNING GUIDELINES

GUIDELINES

- **Scan spacing:** $\leq 1,0$ mm
- **Field of View:** Set Field of View to include entire skull
- **Gantry Tilt:** 0° (if possible)
- **File Type:** DICOM
- **Series:** Original/Primary/Axial (no reconstructed, reformatted or post processed data)
- **Artifacts:** Ensure scans are free from motion artifacts and minimize artifacts from presence of metal
- **Age of scans:** Avoid scans older than 6 months for adult patients

SUBMISSION OF CT SCANS

Online: Make a new folder on your computer's hard drive with patient's ID number as name of folder. Put CT data in folder. Compress folder (ZIP). When you have received confirmation that an account has been created: Log in at: www.ossdsign.com/order and upload the compressed CT data to your order folder.

Postal submission: Mark CD or DVD disc with the patient ID number. Complete the order form and send by post to OssDsign delivery address.

Delivery Address:
Virdings Allé 2
SE 754 50 Uppsala
Sweden

Contact:
+46(0)18-55 39 93
info@ossdsign.com
www.ossdsign.com