

# CT SCANNING GUIDELINES

## GUIDELINES

- **Scan spacing:**  $\leq 1,0$  mm is recommended. Spacing up to 1,25mm can be utilized if the ordering surgeon approves the potential risk that the implant may not fit optimally.
- **Age of scans:** Scans  $\leq 6$  months is recommended. Scans between 6-12 months can be utilized if the ordering surgeon approves the potential risk that the implant may not fit optimally.
- **Field of View:** Set Field of View to include entire skull
- **Gantry Tilt:**  $0^\circ$  (if possible)
- **File Type:** DICOM
- **Series:** Original/Primary/Axial (no reconstructed, reformatted or post processed data)
- **Artefacts:** Ensure scans are free from motion artefacts and minimize artefacts from presence of metal.

## SUBMISSION OF CT SCANS

**Online submission:** You can submit your CT scans via the OssDsign online order system CCP (Customer Communication Platform) together with case specific details. Login to your account via our website or at [ccp.ossdsign.com](http://ccp.ossdsign.com)

**Postal submission:** CT scans can be sent on a disc to OssDsign's delivery address below, together with the order request form. Please mark the disc with a patient identifier.

### Delivery address:

OssDsign AB  
Rapskatan 23 A  
SE - 754 50 Uppsala  
Sweden